Extraction buffer (10x)

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Extraction buffer (10x)

Extraction buffer (10x) for the extraction of pancreatic elastase from stool samples of human origin for use with the IVD product Pancreatic Elastase ELISA

REF	ACS-002	1 pc.
IVD	In-vitro-diagnostic medical device	CE

Seramun Diagnostica GmbH • Spreehagener Str. 1 • 15754 Heidesee • Germany •
T +49 33767 791-10 • info@seramun.com • www.seramun.com

IVD In-vitro diagnostic medical device

 Country of manufacture and date of manufacture

 Keep away from sunlight

 Consult instructions for use

 Sufficient for n tests

UDI Unique device identifier

REF Article number

 Humidity limitation

 Temperature limit

 Biohazard

 Manufacturer

SN Serial number

LOT Batch code

 Do not reuse

 Use-by date

 Attention

Intended Use

Extraction buffer (10x) is an IVD accessory for the extraction of pancreatic elastase from stool samples of human origin for use with the IVD product Pancreatic Elastase ELISA by a laboratory professional user.

Extraction buffer (10x) must not be used with specimen materials other than stool samples of human origin, in the near patient setting and by lay persons.

Principle of the Test

The used principle of the test in which the Extraction buffer (10x) is used is explained in more detail in the instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

Test Components (Delivery Scope)

1	EXTRBUF (10x)	Extraction buffer (10x)	50 mL concentrate, for 500 mL solution, colorless, black cap
2		Instructions for Use	1 piece

Additional Materials and Aids Required for the Test Procedure

Adjustable single-channel-micropipette • Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS) • deionized water • measuring cylinder • tubes for sample preparation • precision scale for sample weighing

Important Information

This component is for *in-vitro* diagnostic use only. Follow the instructions carefully. The component may be used by laboratory professionals only. Do not use reagents from damaged packages or bottles. Imprinted expiry dates must be observed. Do not mix components with reagents from other manufacturers.

Mixing of components of different lots is not allowed.

All serious incidents occurring in relation with Extraction buffer (10x) must be reported to the manufacturer and the competent authority of the EU member state in which user and/or patient are located.

Information on Assay Procedure

Reagent should be stored at 2...8 °C. Bring component to room temperature before use. Reagent that appear contaminated should not be used.

Follow the pipetting scheme and time schedules of the protocol.

Each sample and control should be pipetted with a new pipette tip.

Safety Instructions

Reagent must not be swallowed. Contact with skin or mucous membranes should be avoided.

Handle component and patient samples as if potentially hazardous and infectious.

Additional information may be taken from the Material Safety Data Sheet.

Product contains the following hazardous component/-s:

Component	Number	Full text
EXTRBUF (10x)	-	-

Limitations of the Procedure

The Extraction buffer (10x) is to be used in combination with the IVD device Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

At temperatures higher than 40 °C, samples should be transported refrigerated or frozen (see Sample Shelf Life and Storage). Aqueous stool samples may result in falsely low results due to the dilution effect. In case of watery stool, another sample with more solid consistency should be obtained. If this is not possible, stool may be pipetted (e.g. 10 µL watery stool + 1 mL Extraction buffer (1x)). Since there is a risk of false pathological (falsely low) values due to the dilution of the stool, the consistency should be noted and in case of a pathological finding, the test must be repeated with a solid stool sample.

Sample Treatment

Sample Collection

Collect stool sample in suitable sampling container.

Sample Shelf Life and Storage

Stool samples can be stored at different temperatures for the following time periods.

<u>Native human stool sample</u>		<u>Extracted sample supernatant</u>	
storage	duration	storage	duration
2...8 °C	7 days	2...8 °C	3 days
20...22 °C	5 days	-18...-27 °C	1 month
40 °C	2 days		

Stool samples and extracted sample supernatants may be frozen and thawed up to two times. It is recommended to aliquot samples to avoid frequent freeze and thaw cycles.

Sample Preparation

For sample weighing:

1. Preparation of Extraction buffer (10x) (G09038P01/ACS-002) by 1 : 10 dilution (= 1+9) with deionized water (e.g. 50 mL Extraction buffer (10x) + 450 mL deionized water)
2. Mixing of stool samples of human origin 1 : 101 with Extraction buffer (1x) (e.g. 10 mg stool + 1 mL Extraction buffer (1x)).
3. Homogenization of the samples by means of a rotary shaker and subsequent sedimentation of solid constituents.
4. Removal of the supernatants after 15 - 30 min sedimentation at room temperature or alternatively at 2...8 °C overnight.

Note: If salt crystals have precipitated in the Extraction buffer, dissolve them before use by gently shaking and heating in a water bath at 30 - 35 °C. For analysis, the collected supernatants can be stored at -18 °C for up to 4 weeks. It is important that supernatants are separated from the sediment during storage.

Note: Further sample treatment is performed according to the instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

Reagent Treatment

Reagent Shelf Life and Storage

The Extraction buffer (10x) with sealed reagent bottles can be stored at 2...8 °C until the printed expiration date. The diluted extraction buffer can be used at 2...8 °C up to 1 month.

Reagent Preparation

Bring Extraction buffer (10x) to room temperature before use.

Dilute Extraction buffer (10x) 1 : 10 with deionized water.

Example: 10 mL Extraction buffer (10x) + 90 mL deionized water.

Assay Procedure

Note: The assay procedure is performed according to the instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

Evaluation of Results

Note: The evaluation of results is performed according to the instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

Interpretation of Results

Note: The interpretation of results is performed according to the instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

Performance Characteristics

Note: The performance characteristics of the Extraction buffer (10x) have been determined in combination with the IVD device Pancreatic Elastase ELISA. Please refer to instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS) for more information on the performance characteristics.

Change History

Version	Section	Modifications
2022-09_v01_DE_EN	Entire document	Updating of the intended use Conversion of subsections Insertion of safety instructions

References

Please refer to instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS) for more information on the references.