Serazym[®] ASCA IgA

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Lagertemperatur

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Serazym[®] ASCA IgA

Enzyme immunoassay for detection of IgA antibodies to *Saccharomyces cerevisiae*
in human serum or plasma

REF E-032  96  *In-vitro*-diagnostic device 

Seramun Diagnostica GmbH · Sprenhagener Straße 1 · 15754 Heidesee · Germany · www.seramun.com
phone +49 (0) 33767 79110 · fax +49 (0) 33767 79199 · info@seramun.com

Introduction

Non-specific inflammatory bowel diseases including Crohn's disease (Enteritis regionalis) and ulcerative colitis (UC) are characterized by unknown etiology as well as chronic-remitting inflammatory processes of the intestine.

Whereas the inflammation of ulcerative colitis is restricted to the mucosa and submucosa of colon and rectum, Crohn's, disease (CD) shows a wide spread inflammation of the gastro-intestinal tract with granuloma formation. The risk developing one of these diseases is strongly influenced by immunologic, genetic, infectious and environmental factors. The differential diagnosis of inflammatory bowel diseases to chronic diarrhea, recurrent abdominal dolor, infectious colitis, anorexia as well as the differentiation of CD to ulcerative colitis is still a high challenge.

The determination of IgA and IgG antibodies to *Saccharomyces cerevisiae* (baker's yeast) has been described as one important serological marker for the differential diagnosis of Crohn's disease recently. Up to 70 % of patients with CD show antibody levels to *Saccharomyces cerevisiae*. Although the cause for their occurrence has been unclear, antibodies to *Saccharomyces cerevisiae* (ASCA) are strongly associated with inflammatory processes of the intestine. In combination with the detection of autoantibodies to atypical antineutrophil cytoplasmic antigens (aANCA) which are mainly found in patients with ulcerative colitis, ASCA are a valid parameter for the differentiation of Crohn's disease and ulcerative colitis.

References:

1. Conrad K, Schmechta H, Klafki A, Lobeck G, Uhlig HH, Gerdi S, Henker J: Serological differentiation of inflammatory bowel diseases. Eur J Gastrol & Hepatol. 2002 14:129-135
2. Vermeire S: Serological Diagnosis in IBD. IBDM 2002 3:82-89

Intended use

Serazym® ASCA IgA is an *in-vitro* diagnostic device for quantitative determination of IgA antibodies to mannan of the yeast *Saccharomyces cerevisiae* in human serum or plasma.

Principle of the test

In the first incubation step diluted samples, ready-to-use calibrators and positive control react with the solid-phase adsorbed antigen. Unbound components are removed after 60 minutes incubation at room temperature (RT, 18...25 °C) by aspirating the samples followed by a washing step.

In the second incubation step bound horseradish peroxidase (HRP)-labelled anti-human-IgA antibodies to sample antibodies. Unbound conjugate is removed after 30 minutes incubation at RT by aspirating followed by a washing step.

In the third step the HRP converts the subsequently added colourless substrate solution of hydrogen peroxide and 3,3',5,5'-tetramethylbenzidine (TMB) within a 15 min reaction time at RT into a blue product. The enzyme reaction is terminated by sulphuric acid dispensed into the wells turning the solution from blue to yellow.

The absorbance of the solution read at 450/≥620 nm (OD) is directly proportional to the amount of specifically bound antibodies. A calibration curve is established by plotting the concentrations of the antibodies of the calibrators (x-axis) and their corresponding absorbance values (y-axis). The antibody concentrations of the unknown specimens are directly read from the calibration curve.

Test components

		For 96 Wells
1	WELLS	Microtitration plate coated with antigen
		12 single breakable 8-well strips vacuum-sealed with desiccant
2	WASHBUF CONC 10x	Wash buffer 10-fold
		100 ml concentrate for 1000 ml solution white cap
3	DIL	Sample diluent
		100 ml · ready-to-use coloured red black cap
4	CAL 0 - 4	Calibrator 0 - 4 diluted serum samples CAL 0 = 1 U/ml CAL 1 = 10 U/ml CAL 2 = 30 U/ml CAL 3 = 100 U/ml CAL 4 = 300 U/ml
		1.0 ml per calibrator ready-to-use coloured blue white cap
5	CONTROL +	Positive control diluted serum sample concentration see certificate of analysis enclosed
		1.0 ml · ready-to-use coloured blue red cap
6	CONJ HRP	HRP-conjugate HRP-labelled, anti-human-IgA antibodies
		15 ml · ready-to-use coloured violet violet cap
7	SUBSTR TMB	Substrate 3,3',5,5'-Tetramethylbenzidine and hydrogen peroxide
		15 ml · ready-to-use blue cap
8	STOP	Stop solution 0.25 M sulphuric acid
		15 ml · ready-to-use yellow cap
9		Adhesive film
		2

Preparation and storage of samples

Collection and storage

Blood is taken by venipuncture. Serum or plasma is separated after clotting by centrifugation. Serum or plasma can be used in the *Serazym*[®] ASCA IgA. Contaminated samples should not be used. Repeated freezing and thawing should be avoided. The samples may be kept at 2...8 °C for up to two days. Long-term storage requires -20 °C.

Preparation before use

Allow samples to reach RT prior to assay. Take care to agitate samples gently in order to ensure homogeneity.

Note: Patient samples have to be diluted 1 : 101, e.g. 10 µl sample + 1000 µl sample diluent (3).

Materials required but not provided

Adjustable one-channel micropipette 0.100 - 1.000 ml and 0.010 - 0.100 ml · Adjustable 8-channel micropipette 0.050 - 0.200 ml · Pipette tips · Reagent containers for dispensing with 8-channel pipette · Graduated measuring flasks 10 ml and 100 ml · Test tubes 2.0 ml for sample dilution · Microtitration plate washer (automatic or hand wash head) · Microtitration plate reader with optical filters of 450 nm for measurement and ≥ 620 nm for reference · Distilled or deionized water · Stop-watch

Preparation and storage of reagents

Kit size and expiry

The *Serazym*[®] ASCA IgA has been designed for 12 x 8 determinations.

The complete kit with unopened reagent bottles and microtitration strips is stable until the expiry date printed on the kit box in case of storage at 2...8 °C. After opening all kit components are stable for at least 2 months, provided proper storage. The ready-to-use wash solution can be used for at least one month when stored at 2...8°C.

Reagent preparation

The microtitration plate is vacuum-sealed in a foil with desiccant. The plate consists of a frame and strips with breakable wells. Allow the sealed plate to reach room temperature before opening. Unused wells should be stored refrigerated and protected from moisture in the original cover carefully resealed.

Prepare a sufficient amount of wash solution by diluting the 10-fold concentrated wash buffer 1 + 9 with distilled or deionized water.

For example: 10 ml wash buffer (2) + 90 ml distilled or deionized water.

Besides wash buffer all components of the device are ready-to-use.

Assay procedure

Allow all components to reach room temperature prior to use in the assay.

Dilute patient samples with sample diluent (3) 1 : 101 (v/v), e.g. 10 µl sample + 1000 µl sample diluent (3).

The calibrators and the positive control are ready-to-use.

Avoid any time shift during dispensing of reagents and samples.

Make sure the soak time of the wash buffer in the wells is at least 5 seconds per wash cycle and that the remaining fluid is completely drained in every single wash cycle!

Avoid light exposure of the TMB substrate solution!

Working steps

1. Warm all reagents to room temperature (RT) before use. Mix gently without causing foam.
2. Pipette: 100 µl **CAL 0** calibrator 0 (4) if desired*,
 100 µl **CAL 1 - 4** calibrators 1, 2, 3, 4 (4)
 100 µl **CONTROL +** positive control (5)
 100 µl **diluted samples** resp.
 into the intended wells.
3. Cover plate (9) and incubate for 60 min at RT.
4. Decant, then wash each well 3x with 300 µl wash solution (diluted from (2)) and tap dry onto absorbent paper.
5. Dispense 100 µl **CONJ HRP** HRP-conjugate (6) per well.
6. Cover plate (9) and incubate for 30 min at RT.
7. Decant, then wash each well 3x with 300 µl wash solution (diluted from (2)) and tap dry onto absorbent paper.
8. Dispense 100 µl **SUBSTR TMB** substrate (7) per well.
9. Incubate for 15 min at RT protected from light.
10. Dispense 100 µl **STOP** stop solution (8) per well, mix gently.
11. Read OD at 450 nm / \geq 620 nm with a microplate reader within 30 min after reaction stop.

* Note:

Using CAL 0 (1 U/ml) enables quantification of concentrations below the value of CAL 1 (10 U/ml), otherwise CAL 0 is not needed.

Result interpretation

If desired create a calibration curve by plotting the mean absorbances of the calibrators CAL 0-4 (Y-axis) to their corresponding antibody-concentrations (X-axis).
 Alternatively create a calibration curve by plotting the mean absorbances of the calibrators CAL 1-4 (Y-axis) to their corresponding antibody-concentrations (X-axis).
 For the calibration curve use the 4-Parameter-regression-model.

Determine the antibody-concentrations of the unknown samples by referring their mean absorbances to the calibration curve. In case of using the prescribed sample dilution of 1 : 101, antibody-concentrations are not to be determined by multiplication with the dilution factor, since the calibrators are already prediluted.

If a sample after proper dilution of 1:101 produced a higher absorbance than the Calibrator CAL 4 please retest the sample at higher dilution and calculate its concentration in the following way:

Sample volume	Added volume of diluent	Final volume	Dilution of original sample	Multiplication factor to determine analyte concentration of the original sample
10µl	1000µl	1010µl	1 : 101 = Sample 1	<i>No multiplication required</i>
500µl Sample 1	500µl	1000µl	1 : 202 = Sample 2	factor 2
500µl Sample 2	500µl	1000µl	1 : 404 = Sample 3	factor 4
500µl Sample 3	500µl	1000µl	1 : 808 = Sample 4	factor 8

Please be aware that antibodies may not be stable in diluted samples resulting in activity loss. By that reason reinvestigation at higher dilution should be done at the same day.

Reference values

Serazym® ASCA IgA	
positive	≥ 20 U/ml
negative	< 20 U/ml

Test validity

The test run is valid if:

- absorbance of calibrator CAL 0 is < CAL 1
- absorbance of calibrator CAL 1 is ≤ 0.50
- absorbance of calibrator CAL 4 is ≥ 1.20

If the above mentioned quality criteria are not met, repeat the test and make sure that the test procedure is followed correctly (incubation volumes, times and temperatures, sample and wash buffer dilution, wash steps etc.). In case of repeated failure of the quality criteria contact your supplier.

Limitations of the method

Any clinical diagnosis should not be based on the results of *in-vitro* diagnostic methods alone. Physicians are supposed to consider all clinical and laboratory findings possible to state a diagnosis. False results may be caused by cross contaminations of the kit reagents and samples, bacterial or fungal contaminations of reagents and/or samples, incorrect washing and incorrect incubation times. In case of background problems when using automatic microplate ELISA systems it may be recommendable to wash wells 5 times (instead of 3) in every wash cycle.

Performance characteristics

Diagnostic sensitivity

A total of 82 serum samples from patients with Crohn's disease, 65 serum samples from patients with ulcerative colitis, 101 serum samples from patients with celiac disease, 33 serum samples from patients with PBC, 44 serum samples from patients with SLE were tested in the *Serazym®* ASCA IgA.


In reference to this panel the diagnostic sensitivity was determined 50 %.

Diagnostic specificity

Diagnostic specificity of the *Serazym®* ASCA IgA was determined 94 % by testing n=250 serum samples from healthy blood donors.

Frequency distribution

The frequency distribution of the ASCA IgA antibody concentrations measured in 84 samples is illustrated: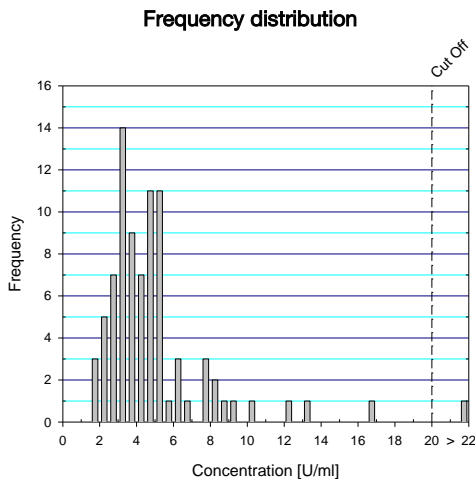

Precision


Intra-assay coefficient of variation (CV [%]) in the *Serazym*[®] ASCA IgA from 8-fold determinations of samples:

sample	concentration [U/ml]	CV [%]
1	205	2.39
2	167	3.56
3	115	2.23
4	76	2.00

Inter-assay coefficient of variation (CV [%]) in the *Serazym*[®] ASCA IgA in 16 different test runs from threefold determination of samples:

sample	concentration [U/ml]	CV [%]
A	209	11.38
B	174	8.28
C	113	7.20
D	72	5.92

Typical reference curve

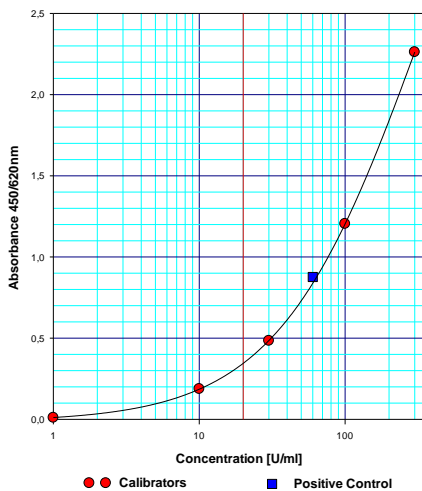Common advices and precautions

This kit is for *in-vitro* use only. Follow the working instructions carefully. The kit should be performed by trained technical staff only. The expiration dates stated on the respective labels are to be observed. The same relates to the stability stated for diluted reagents. Do not use or mix reagents from different lots, damaged packages or bottles or reagents from other manufacturers. Avoid time shift during dispensing of reagents. All reagents should be kept at 2...8 °C before use.

Do not use or mix reagents from different lots except for sample diluent, wash buffer, substrate and stop solution.

Some of the reagents contain small amounts of Thimerosal (< 0.01% w/v) and Kathon (1.0% v/v) as preservative. They may neither be swallowed nor come into contact with skin or mucous membranes. In case of contact rinse with plenty of water. Handle all samples as potentially infectious. Since the kit contains potentially hazardous materials, the following precautions must always be observed:

Do not smoke, eat or drink while handling kit material!

Always use protective gloves!

Never pipette material by mouth!

Note safety precautions of the single test components!

Incubation scheme *Serazym*[®] ASCA IgA

- | | | | |
|----|---|----------|--|
| 1. |  | 100 µl | CAL 0 - 4 (4) |
| | | 100 µl | CONTROL + (5) |
| | | 100 µl | diluted samples resp. |
| |  | 60 min | incubation (room temperature) |
| | | 3 x wash | with wash solution |
| 2. |  | 100 µl | CONJ HRP (6) |
| | | 30 min | incubation (room temperature) |
| |  | 3 x wash | with wash solution |
| 3. |  | 100 µl | SUBSTR TMB (7) |
| | | 15 min | incubation (room temperature) protected from light |
| 4. |  | 100 µl | STOP (8) |

Read OD at 450 / \geq 620 nm

Manufacturer

Catalogue-No.

Lot-No.

Number of determinations

Biohazard

Notice advices

Use by

Storage temperature

Consult instructions for use