# Serazym<sup>®</sup> Anti-Transglutaminase IgA

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# Serazym<sup>®</sup> Anti-Transglutaminase IgA

Enzyme immunoassay for the quantitative determination of IgA antibodies against tissue transglutaminase in serum or plasma of human origin

<b>REF</b>	E-047		96
<b>IVD</b>	In-vitro-diagnostic medical device		

**Seramun Diagnostica GmbH** • Spreenhagener Str. 1 • 15754 Heidesee • Germany •  
 T +49 33767 791-10 • [info@seramun.com](mailto:info@seramun.com) • [www.seramun.com](http://www.seramun.com)

<p><b>IVD</b> In-vitro diagnostic medical device</p> <p> Country of manufacture and date of manufacture</p> <p> Keep away from sunlight</p> <p> Consult instructions for use</p> <p> Sufficient for <i>n</i> tests</p>	<p><b>UDI</b> Unique device identifier</p> <p><b>REF</b> Article number</p> <p> Humidity limitation</p> <p> Temperature limit</p> <p> Biohazard</p>	<p> Manufacturer</p> <p><b>SN</b> Serial number</p> <p><b>LOT</b> Batch code</p> <p> Do not reuse</p> <p> Use-by date</p> <p> Attention</p>
--	--	---

## Intended Use

Serazym® Anti-Transglutaminase IgA is an IVD test for the quantitative determination of antibodies of the IgA isotype against tissue transglutaminase in serum or plasma (citrate, EDTA, heparin) of human origin by a laboratory professional user.

It is intended to aid in the diagnosis and monitoring of celiac disease in samples from patients with symptoms of celiac disease and for screening of antibodies of the IgA isotype against tissue transglutaminase in certain at-risk groups.

The test must not be used with specimen materials other than serum or plasma (citrate, EDTA, heparin) of human origin, for diagnosis, prediction, prognosis, as companion diagnostic, in the near-patient setting and by lay persons.

## Principle of the Test

Serazym® Anti-Transglutaminase IgA is an enzyme immunoassay in which diluted serum or plasma samples, ready-to-use calibrators and positive control react with the antigen (tissue transglutaminase and gliadin) adsorbed to the solid phase. After a 60-minutes incubation at room temperature (RT, 18...25 °C) unbound components are removed by a wash step. In the second incubation step horseradish peroxidase (HRP)-labeled anti-human IgA antibodies bind to the sample antibodies. After a 30-minutes incubation at RT unbound conjugate is removed by a wash step. HRP converts the colorless substrate solution to a blue reaction product in the following 15-minutes enzymatic reaction step. The reaction is stopped by addition of the stop solution, resulting in a color change from blue to yellow. The optical density (OD) of the reaction product measured at 450 nm measuring and  $\geq 620$  nm reference filter, respectively, is directly proportional to the concentration of specifically bound antibodies. Using the measured absorbances of the calibrators and their corresponding antibody concentrations, a calibration curve is generated from which the concentrations of the patient samples can be read.

## Test Components (Delivery Scope)

			<b>For 96 wells</b>
1	<b>WELLS</b>	<b>Microtiter plate</b> coated with tissue transglutaminase and gliadin	12 single breakable 8-well strips, colorless, vacuum-sealed with desiccant
2	<b>WASHBUF (10x)</b>	<b>Wash buffer (10x)</b> Seramun® Wash buffer K TRIS based buffer	100 mL concentrate for 1000 mL solution, colorless, white cap
3	<b>DIL</b>	<b>Sample diluent</b> Seramun® Sample diluent L TRIS based buffer	100 mL, ready to use, colored red, white cap
4	<b>CAL</b>   <b>0 - 4</b>	<b>Calibrators 0 – 4</b> <b>CAL 0 = 1 U/mL</b> <b>CAL 1 = 10 U/mL</b> <b>CAL 2 = 30 U/mL</b> <b>CAL 3 = 100 U/mL</b> <b>CAL 4 = 300 U/mL</b>	1.0 mL each ready-to-use colored blue white cap
5	<b>CONTROL</b>   <b>+</b>	<b>Positive control</b> Concentration see Certificate of Analysis	1.0 mL, ready to use, colored blue, red cap
6	<b>CONJ HRP</b>	<b>HRP conjugate</b> HRP labeled, anti-human-IgA antibody (sheep)	15 mL, ready to use, colored violet, violet cap
7	<b>SUBSTR</b>	<b>Substrate</b> SeramunBlau® slow2 70 < 0.1 % 3,3',5,5'-tetramethylbenzidine	15 mL, ready to use, colorless, blue cap

8	<b>STOP</b>	<b>Stop solution</b> SeramunBlau® stop 0.25 M sulphuric acid	15 mL, ready to use, colorless, yellow cap
9	<b>COVER</b>	<b>Covering film</b>	2 pieces
10		<b>Certificate of Analysis</b>	1 piece
11		<b>Instructions for Use</b>	1 piece

## Additional Materials and Aids Required for the Test Procedure

Adjustable single-channel micropipette • 8-channel pipette or multi-pipette with pipette tips • reagent container for multi-channel micro-pipettes • 8-channel wash comb with vacuum pump and waste bottle or microplate washer • microplate reader with 450 nm measuring filter and  $\geq 620$  nm reference filter • deionized water • measuring cylinder • tubes for sample preparation

## Important Information

**This device is for *in-vitro* diagnostic use only.** Follow the instructions carefully. The kit may be performed by health professionals only.

Do not use reagents from damaged packages or bottles. The shelf life specified must be observed. Do not mix components with reagents from other manufacturers.

**Mixing of test kit components of different lots is only allowed for wash buffer (10x), substrate and stop solution.**

All serious incidents occurring in relation with Serazym® Anti-Transglutaminase IgA must be reported to the manufacturer and the competent authority of the EU member state in which user and/or patient are located.

### Information on Assay Procedure

All reagents should be stored at 2...8 °C. Bring all test components to room temperature before use. For larger sample series, pipetting reagents from liquid reservoirs using a multichannel pipette is recommended to avoid time delays. Follow the pipetting scheme and time schedules of the protocol. The aspiration and washing steps can be performed manually or with the help of a microplate washer or waterjet pump. Wash solution should be allowed a minimum reaction time of 5 s in the wells per wash cycle. Remove wash buffer residues by thoroughly aspirating or tapping out the cavities! Protect substrate from light!

### Safety Instructions

Reagents must not be swallowed. Contact with skin or mucous membranes should be avoided. Some reagents may contain biocides as preservative. Handle all components and patient samples as if potentially hazardous and infectious.

Additional information may be taken from the Material Safety Data Sheet.

Product contains the following hazardous component/-s:

<b>WELLS</b>	-	Contains material of animal origin.
<b>WASHBUF (10x)</b>	EUH208	Contains reaction mass of 5-chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.
	EUH210	Safety data sheet available on request.
<b>DIL</b>	-	Contains material of animal origin.
<b>CAL</b> 0 - 4	EUH208	Contains reaction mass of 5-chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.
	EUH210	Safety data sheet available on request.
<b>CONTROL</b> +	EUH208	Contains reaction mass of 5-chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.
	EUH210	Safety data sheet available on request.
<b>CONJ HRP</b>	Hazard components H360D P202	N-methyl-2-pyrrolidone; 1-methyl-2-pyrrolidone May damage the unborn child. Do not handle until all safety precautions have been read and understood.
	P280	Wear protective gloves/protective clothing/eye protection/face protection.
	P308+P313	IF exposed or concerned: Get medical advice/attention.
	P501	Dispose of contents/container in accordance with local regulations.
	EUH208	Contains reaction mass of 5-chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.  Restricted to professional users.  Contains material of animal origin.
<b>SUBSTR</b>	EUH210	Safety data sheet available on request.
<b>STOP</b>	Hazard components H290	Sulphuric acid 2.5 % May be corrosive to metals.

### Limitations of the Procedure

As with all immunological determination methods, contamination of the reagents, but also of the samples by bacteria or fungi, as well as cross-contamination of the test kit reagents and samples can lead to incorrect results. Incorrect washing to separate unbound components from the sample and test reagents as well as incorrect incubation times of the samples and calibrators can cause incorrect results.

The overall interpretation of the ELISA test result should be made in the context of the clinical findings. False negative results may occur in patients with IgA deficiency. For this reason, a test for celiac disease specific IgG antibodies is recommended.

## Sample Treatment

### Sample Collection

Collect serum or plasma (citrate, EDTA, heparin) of human origin in suitable sampling container.

### Sample Shelf Life and Storage

Samples can be stored at 2...8 °C for a maximum of 2 days. For longer periods samples have to be stored at -20 °C. Repeated (> 3x) freezing and thawing of samples should be avoided.

### Sample Preparation

Bring samples to room temperature before use. Homogeneity should be ensured by shaking components briefly. Samples have to be diluted 1 : 101 (v/v) with sample diluent.

Example: 10 µL sample and 1000 µL sample diluent.

## Reagent Treatment

### Reagent Shelf Life and Storage

The complete test kit with sealed reagent bottles and microtitration strips can be stored at 2...8 °C until the printed expiration date. All opened test kit components are stable for up to 2 months when stored properly at 2...8 °C. The diluted wash buffer can be stored at 2...8 °C for up to 1 month.

### Reagent Preparation

The Microtiter plate with breakable 8-well strips is vacuum sealed with desiccant. Allow the sealed plate to reach room temperature before opening. Unused wells should be stored at 2...8 °C and protected from moisture in the original cover carefully resealed.

Dilute wash buffer (10x) 1 : 10 with deionized water.

Example: 10 mL wash buffer (10x) + 90 mL deionized water.

Except for the wash buffer, all components included in the test kit are ready to use.

The substrate must be protected from direct light. The test should not be performed with a substrate solution that is colored dark or contains colored precipitates.

## Assay Procedure

1. Allow test reagents and required number of wells to reach room temperature (RT). Shake reagents gently before use. Avoid foaming.
2. Pipette 100  $\mu\text{L}$   Calibrator 0 optional\*  
 100  $\mu\text{L}$    Calibrators 1, 2, 3 and 4  
 100  $\mu\text{L}$    Positive control  
 100  $\mu\text{L}$  diluted sample each.
3. Cover the plate and incubate for 60 min at RT.
4. Decant, then wash each well 3 x with 300  $\mu\text{L}$  diluted wash buffer. Tap dry onto absorbent paper if necessary.
5. 100  $\mu\text{L}$   HRP conjugate per well.
6. Cover the plate and incubate for 30 min at RT.
7. Decant, then wash each well 3 x with 300  $\mu\text{L}$  diluted wash buffer. Tap dry onto absorbent paper if necessary.
8. 100  $\mu\text{L}$   substrate per cavity.
9. Incubate for 15 min at RT **protected from light**.
10. 100  $\mu\text{L}$   stop solution per cavity, mix gently.
11. Read OD at 450 nm and  $\geq 620$  nm with a microplate reader within 30 min following reaction stop.

\* Note:

Using CAL 0 (1 U/mL) allows quantification of concentrations below the CAL 1 value (10 U/mL). If no quantification below the CAL 1 value is required, CAL 0 can be considered optional.

## Evaluation of Results

A calibration curve is created by plotting the measured absorbances of the calibrators CAL 0 - 4 (y-axis) against the corresponding antibody concentrations (x-axis). Alternatively, a calibration curve can be created by plotting the measured absorbances of calibrators CAL 1 - 4 (y-axis) against the corresponding antibody concentrations (x-axis).

The 4-parameter regression model is recommended to create the calibration curve.

The absorbances of the diluted samples are converted to antibody concentrations U/mL by extrapolating the calibration curve. The indicated concentrations of the calibrators already take into account the 1 : 101 dilution of the serum samples.

Measurements of samples with a higher absorbance than that of calibrator CAL 4 have to be repeated in a higher predilution. Results have to be multiplied with the factor of the additional dilution:

Sample volume	Added volume of sample diluent	Final volume	Dilution of original sample	Multiplication factor
10 $\mu\text{L}$	1000 $\mu\text{L}$	1010 $\mu\text{L}$	1 : 101 = <b>Sample 1</b>	-
500 $\mu\text{L}$ <b>Sample 1</b>	500 $\mu\text{L}$	1000 $\mu\text{L}$	1 : 202 = <b>Sample 2</b>	<b>2</b>
500 $\mu\text{L}$ <b>Sample 2</b>	500 $\mu\text{L}$	1000 $\mu\text{L}$	1 : 404 = <b>Sample 3</b>	<b>4</b>

Note, antibodies in dilutions may not be stable, which may lead to a loss of activity. For this reason, a reinvestigation should be performed in a higher dilution at the same day.

## Interpretation of Results

Positive	$\geq 20$ U/mL
Negative	$< 20$ U/mL

### Test validation

The test run is valid if

- adsorbance of calibrator CAL 0  $<$  CAL 1
- adsorbance of calibrator CAL 1  $\leq 0.50$
- adsorbance of calibrator CAL 4  $\geq 1.20$

If the above-mentioned quality criteria are not met, test should be repeated strictly following the test procedure (reagent preparation, incubation times and temperatures, wash steps, etc.). In case of repeated failure of the quality criteria contact the manufacturer.

## Performance Characteristics

### Sensitivity and Specificity

35 serum samples from celiac disease patients and 170 serum samples from blood donors have been tested in Serazym® Anti-Transglutaminase IgA and analyzed by ROC analysis.

At a cut-off of 20 U/mL sensitivity was determined at 91.4 % and specificity at 97.1 %.

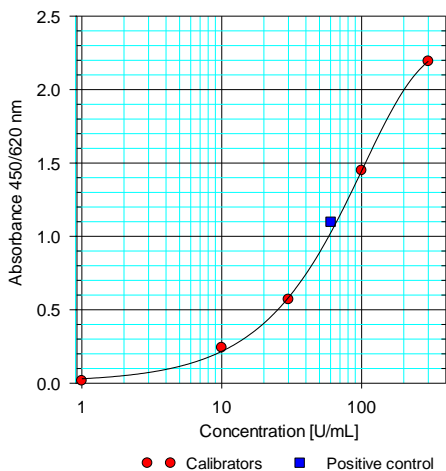
### Precision

To determine precision, 4 samples were measured multiple times. For the determination of the intra-assay coefficient of variation (CV) samples were measured in an 8-fold determination in one test run. The determination of the inter-assay coefficient of variation was done by a 2-fold determination in 16 different test runs:

Sample	Intra-assay coefficient of variation		Inter-assay coefficient of variation	
	Concentration [U/mL]	CV [%]	Concentration [U/mL]	CV [%]
1	206	8.88	231	13.70
2	148	2.75	154	10.19
3	59	3.53	63	8.63
4	29	2.74	27	7.73

## Typical Calibration Curve

A typical calibration curve of Serazym® Anti-Transglutaminase IgA is shown below:

## Interference

Hemolytic, lipemic and icteric samples do not interfere when present in concentrations up to 500 mg/dL (hemoglobin), 1000 mg/dL (lipids), and 20 mg/dL (bilirubin C and bilirubin F). Rheumatoid factors do not interfere when present in concentrations up to 550 IU/mL.

## Change History

Version	Section	Modifications
2022-11_v01_DE_EN	Entire document	Updating of the Intended Use Conversion of subsections Insertion of Safety Instructions Updating Important information

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