# Serazym<sup>®</sup> Anti-MPO (pANCA) IgG

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# Serazym<sup>®</sup> Anti-MPO (pANCA) IgG

Enzyme immunoassay for the quantitative detection of IgG autoantibodies against myeloperoxidase (MPO) in serum or plasma of human origin

|            |                                    |  |    |
|------------|------------------------------------|--|----|
| <b>REF</b> | E-087                              |  | 96 |
| <b>IVD</b> | In-vitro-diagnostic medical device |  |    |

**Seramun Diagnostica GmbH** • Spreehagener Str. 1 • 15754 Heidesee • Germany •  
 T +49 33767 791-10 • [info@seramun.com](mailto:info@seramun.com) • [www.seramun.com](http://www.seramun.com)

|   |                                     |                         |
|---|-------------------------------------|-------------------------|
| <b>IVD</b> In-vitro diagnostic medical device         | <b>UDI</b> Unique device identifier | <b>Manufacturer</b>     |
| <b>Country of manufacture and date of manufacture</b> | <b>REF</b> Article number           | <b>SN</b> Serial number |
| <b>Keep away from sunlight</b>                        | <b>Humidity limitation</b>          | <b>LOT</b> Batch code   |
| <b>Consult instructions for use</b>                   | <b>Temperature limit</b>            | <b>Do not reuse</b>     |
| <b>Sufficient for <i>n</i> tests</b>                  | <b>Biohazard</b>                    | <b>Use-by date</b>      |
|   |                                     | <b>Attention</b>        |

## Intended Use

Serazym® Anti-MPO (pANCA) IgG is an IVD test for the quantitative determination of autoantibodies of the IgG isotype against myeloperoxidase (MPO) in serum or plasma (citrate, EDTA, heparin) of human origin by a laboratory professional user.

It is intended to aid in the diagnosis of antineutrophil cytoplasmic antibodies (ANCA)-associated vasculitis (AAV) in specimen materials from patients with suspicion of a blood vessel disease.

The test must not be used with specimen materials other than serum or plasma (citrate, EDTA, heparin) of human origin, for diagnosis, monitoring, screening, prediction, prognosis, as companion diagnostic, in the near-patient setting and by lay persons.

## Principle of the Test

Serazym® Anti-MPO (pANCA) IgG is an enzyme immunoassay in which the diluted serum or plasma samples, the ready to use calibrators and the positive control react with the antigen adsorbed to the solid phase. After a 60-minute incubation at room temperature (RT, 18...25 °C) unbound components are removed by a wash step. In the second incubation step horseradish peroxidase (HRP) labeled anti-human IgG antibodies bind to the sample antibodies. After a 30-minute incubation at RT unbound conjugate is removed in a wash step. HRP converts the colorless substrate solution to a blue reaction product in the following 15-minute enzymatic reaction step. The reaction is stopped by addition of the stop solution, resulting in a color change from blue to yellow. The optical density (OD) of the reaction product measured at 450 nm measuring and  $\geq 620$  nm reference filter, respectively, is directly proportional to the concentration of specifically bound antibodies. Using the measured absorbances of the calibrators and their corresponding antibody concentrations, a calibration curve is generated from which the concentrations of the patient samples can be read.

## Test Components (Delivery Scope)

|   |                           |   | <b>For 96 wells</b>  |
|---|---------------------------|---|--|
| 1 | <b>WELLS</b>              | <b>Microtiter plate</b><br>coated with myeloperoxidase (MPO)  | 12 single breakable 8-well strips, colorless, vacuum-sealed with desiccant |
| 2 | <b>WASHBUF (10x)</b>      | <b>Wash buffer (10x)</b><br>Seramun® Wash buffer K<br>TRIS based buffer   | 100 mL concentrate for 1000 mL solution, colorless, white cap              |
| 3 | <b>DIL</b>                | <b>Sample diluent</b><br>Seramun® Sample diluent M<br>TRIS based buffer   | 100 mL, ready to use, colored red, black cap                               |
| 4 | <b>CAL</b>   <b>0 - 4</b> | <b>Calibrators 0 – 4</b><br><b>CAL 0 = 1 U/mL</b><br><b>CAL 1 = 10 U/mL</b><br><b>CAL 2 = 30 U/mL</b><br><b>CAL 3 = 100 U/mL</b><br><b>CAL 4 = 300 U/mL</b> | 1.0 mL per calibrator ready to use, colored blue, white cap                |
| 5 | <b>CONTROL +</b>          | <b>Positive control</b><br>Concentration see Certificate of Analysis  | 1.0 mL, ready to use, colored blue, red cap                                |
| 6 | <b>CONJ HRP</b>           | <b>HRP conjugate</b><br>HRP labeled, anti-human-IgG antibody (sheep)  | 15 mL, ready to use, colored red, red cap                                  |
| 7 | <b>SUBSTR</b>             | <b>Substrate</b><br>SeramunBlau® slow2 70<br>< 0.1 % 3,3',5,5'-tetramethylbenzidine   | 15 mL, ready to use, colorless, blue cap                                   |
| 8 | <b>STOP</b>               | <b>Stop solution</b><br>SeramunBlau® stop<br>0.25 M sulphuric acid  | 15 mL, ready to use, colorless, yellow cap                                 |

|    |       |                         |          |
|----|-------|-------------------------|----------|
| 9  | COVER | Covering film           | 2 pieces |
| 10 |       | Certificate of Analysis | 1 piece  |
| 11 |       | Instructions for Use    | 1 piece  |

## Additional Materials and Aids Required for the Test Procedure

Adjustable single-channel micropipette • 8-channel pipette or multi-pipette with pipette tips • reagent container for multi-channel micro-pipettes • 8-channel wash comb with vacuum pump and waste bottle or microplate washer • microplate reader with 450 nm measuring filter and  $\geq 620$  nm reference filter • deionized water • measuring cylinder • tubes for sample preparation

## Important Information

**This device is for *in-vitro* diagnostic use only.** Follow the instructions carefully. The kit may be performed by laboratory professionals only.

Do not use reagents from damaged packages or bottles. The shelf life specified must be observed. Do not mix components with reagents from other manufacturers.

**Mixing of test kit components of different lots is only allowed for wash buffer (10x), substrate and stop solution.**

All serious incidents occurring in relation with Serazym® Anti-MPO (pANCA) IgG must be reported to the manufacturer and the competent authority of the EU member state in which user and/or patient are located.

### Information on Assay Procedure

All reagents should be stored at 2...8 °C. Bring all test components to room temperature before use. For larger sample series, pipetting reagents from liquid reservoirs using a multichannel pipette is recommended to avoid time delays. Follow the pipetting scheme and time schedules of the protocol. The aspiration and washing steps can be performed manually or with the help of a microplate washer or waterjet pump. Wash solution should be allowed a minimum reaction time of 5 s in the wells per wash cycle. Remove wash buffer residues by thoroughly aspirating or tapping out the cavities! Protect substrate from light! The test should not be performed with substrate solution that is colored dark or contains colored precipitates.

### Safety Instructions

Reagents must not be swallowed. Contact with skin or mucous membranes should be avoided. Some reagents may contain biocides as preservative. Handle all components and patient samples as if potentially hazardous and infectious.

Additional information may be taken from the Material Safety Data Sheet.

Product contains the following hazardous component/-s:

|                      |                              |  |
|----------------------|------------------------------|--|
| <b>WELLS</b>         | -                            | Contains material of animal origin.  |
| <b>WASHBUF (10x)</b> | EUH208                       | Contains reaction mass of 5-chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.  |
|                      | EUH210                       | Safety data sheet available on request.  |
| <b>DIL</b>           | -                            | Contains material of animal origin.  |
| <b>CAL 0 - 4</b>     | EUH208                       | Contains reaction mass of 5-chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.  |
|                      | EUH210                       | Safety data sheet available on request.  |
| <b>CONTROL +</b>     | EUH208                       | Contains reaction mass of 5-chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.  |
|                      | EUH210                       | Safety data sheet available on request.  |
| <b>CONJ HRP</b>      | Hazard components H360D P202 | N-methyl-2-pyrrolidone; 1-methyl-2-pyrrolidone<br><br>May damage the unborn child.<br>Do not handle until all safety precautions have been read and understood.                    |
|                      | P280                         | Wear protective gloves/protective clothing/eye protection/face protection.   |
|                      | P308+P313 P501               | IF exposed or concerned: Get medical advice/attention.<br>Dispose of contents/container in accordance with local regulations.  |
|                      | EUH208                       | Contains reaction mass of 5-chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.<br><br>Restricted to professional users. |
|                      |                              | Contains material of animal origin.  |
| <b>SUBSTR</b>        | EUH210                       | Safety data sheet available on request   |
| <b>STOP</b>          | Hazard components H290       | Sulphuric acid 2.5 %<br><br>May be corrosive to metals.  |

### Limitations of the Procedure

Contamination of reagents and samples by bacteria or fungi as well as cross-contamination of the test kit reagents and samples can lead to incorrect results. Incorrect washing to separate unbound components from the sample and test reagents as well as incorrect incubation times of the samples, control and calibrators can cause incorrect results as well.

The overall interpretation of an ELISA test result should consider all clinical findings.

## Sample Treatment

### Sample Collection

Collect serum or plasma (citrate, EDTA, heparin) of human origin in suitable sampling container.

### Sample Shelf Life and Storage

Samples can be stored at 2...8 °C for a maximum of 2 days. For longer periods samples have to be stored at -20 °C. Repeated (> 3 x) freezing and thawing of samples should be avoided.

### Sample Preparation

Bring samples to room temperature before use. Homogeneity should be ensured by shaking components briefly. Samples have to be diluted 1 : 101 (v/v) with sample diluent.

Example: 10 µL sample + 1000 µL sample diluent.

## Reagent Treatment

### Reagent Shelf Life and Storage

The complete test kit with sealed reagent bottles and microtitration strips can be stored at 2...8 °C until the printed expiration date. All opened test kit components are stable for up to 2 months when stored properly at 2...8 °C. The diluted wash buffer can be stored at 2...8 °C for up to 1 month.

### Reagent Preparation

The microtiter plate with breakable 8-well strips is vacuum sealed with desiccant. Allow the sealed plate to reach room temperature before opening. Unused wells should be stored at 2...8 °C and protected from moisture in the original cover carefully resealed.

Dilute wash buffer (10 x) 1 : 10 with deionized water.

Example: 10 mL wash buffer (10 x) + 90 mL deionized water.

Except for the wash buffer all components included in the test kit are ready to use.

## Assay Procedure

1. Allow test reagents and required number of wells to reach room temperature (RT). Shake reagents gently before use. Avoid foaming.
2. Pipette 100 µL 

|       |
|-------|
| CAL 0 |
|-------|

 Calibrator 0 optional\*.  
100 µL 

|     |       |
|-----|-------|
| CAL | 1 - 4 |
|-----|-------|

 Calibrators 1, 2, 3, 4.  
100 µL 

|         |   |
|---------|---|
| CONTROL | + |
|---------|---|

 Positive control.  
100 µL diluted sample each, mix gently.
3. Cover the plate and incubate for 60 min at RT.
4. Decant, then wash each well 3 x with 300 µL diluted wash buffer. Tap dry onto absorbent paper if necessary.
5. 100 µL 

|          |
|----------|
| CONJ HRP |
|----------|

 HRP conjugate per well.
6. Cover the plate and incubate for 30 min at RT.
7. Decant, then wash each well 3 x with 300 µL diluted wash buffer. Tap dry onto absorbent paper if necessary.
8. 100 µL 

|        |
|--------|
| SUBSTR |
|--------|

 substrate per well.
9. Incubate for 15 min at RT **protected from light**.
10. 100 µL 

|      |
|------|
| STOP |
|------|

 stop solution per well, mix gently.
11. Read OD at 450 nm and ≥ 620 nm with a microplate reader within 30 min following reaction stop.

\* Note: Using CAL 0 (1 U/mL) allows quantification of concentrations below the CAL 1 value (10 U/mL). If no quantification below the CAL 1 value is required, CAL 0 can be omitted.

## Evaluation of Results

A calibration curve is created by plotting the measured absorbances of the calibrators CAL 0 - 4 (y-axis) against the corresponding antibody concentrations (x-axis). Alternatively, a calibration curve can be created by plotting the measured absorbances of calibrators CAL 1 - 4 (y-axis) against the corresponding antibody concentrations (x-axis).

The 4-parameter regression model is recommended to create the calibration curve.

The absorbances of the diluted samples are converted to antibody concentrations U/mL by extrapolating from the calibration curve. The indicated concentrations of the calibrators already take into account the 1 : 101 dilution of the serum samples.

Measurements of samples with a higher absorbance than that of calibrator CAL 4 have to be repeated in a higher predilution. Results have to be multiplied with the factor of the additional dilution:

| Sample volume          | Added volume of sample diluent | Final volume | Dilution of original sample | Multiplication factor |
|------------------------|--------------------------------|--------------|-----------------------------|-----------------------|
| 10 µL                  | 1000 µL                        | 1010 µL      | 1 : 101 = <b>Sample 1</b>   | -                     |
| 500 µL <b>Sample 1</b> | 500 µL                         | 1000 µL      | 1 : 202 = <b>Sample 2</b>   | <b>2</b>              |
| 500 µL <b>Sample 2</b> | 500 µL                         | 1000 µL      | 1 : 404 = <b>Sample 3</b>   | <b>4</b>              |

Note that antibodies in dilutions may not be stable, which may lead to a loss of activity. For this reason, a reinvestigation should be performed in a higher dilution at the same day.

## Interpretation of Results

|          |                |
|----------|----------------|
| Positive | $\geq 10$ U/mL |
| Negative | $< 10$ U/mL    |

### Test Validation

The test run is valid if:

- absorbance of calibrator CAL 0  $<$  CAL 1
- absorbance of calibrator CAL 1  $\leq 0.70$
- absorbance of calibrator CAL 4  $\geq 1.20$

If the above-mentioned quality criteria are not met, the test should be repeated strictly following the test procedure (reagent preparation, incubation times and temperatures, wash steps, etc.). In case of repeated failure of the quality criteria contact the manufacturer.

## Performance Characteristics

### Sensitivity and Specificity

A total of 81 serum samples were tested in both, Serazym® Anti-MPO (pANCA) IgG and an independent, commercially available ELISA. Results are shown below:

| n = 81                         | ELISA positive | ELISA negative |
|--------------------------------|----------------|----------------|
| <b>Serazym® ELISA positive</b> | 10             | 2              |
| <b>Serazym® ELISA negative</b> | 2              | 67             |

Sensitivity: 83.3 %

Specificity: 97.1 %

### Control Groups

A total of 247 serum samples were tested in Serazym® Anti-MPO (pANCA) IgG. The following 9 control groups were included in this serum panel: rheumatoid arthritis, SLE, scleroderma, MCTD, myositis, Sjögren's syndrome, ulcerative colitis, Crohn's disease, and other collagenoses. 34 serum samples were found to be positive in Serazym® Anti-MPO (pANCA) IgG. This corresponds to a diagnostic specificity of 86.2 %.


### Precision

To determine precision, 4 samples were measured multiple times. For the determination of the intra-assay coefficient of variation (CV) samples were measured in an 8-fold determination in one test run. The determination of the inter-assay coefficient of variation was done by a 2-fold determination in 16 different test runs:

| Sample | Intra-assay coefficient of variation |        | Inter-assay coefficient of variation |        |
|--------|--------------------------------------|--------|--------------------------------------|--------|
|        | Concentration [U/mL]                 | CV [%] | Concentration [U/mL]                 | CV [%] |
| 1      | 69.7                                 | 2.90   | 87.1                                 | 7.29   |
| 2      | 47.8                                 | 2.62   | 48.6                                 | 6.39   |
| 3      | 38.2                                 | 2.84   | 36.5                                 | 6.00   |
| 4      | 18.1                                 | 2.88   | 20.5                                 | 5.88   |

## Calibration curve

A typical calibration curve in Serazym® Anti-MPO (pANCA) IgG is shown below: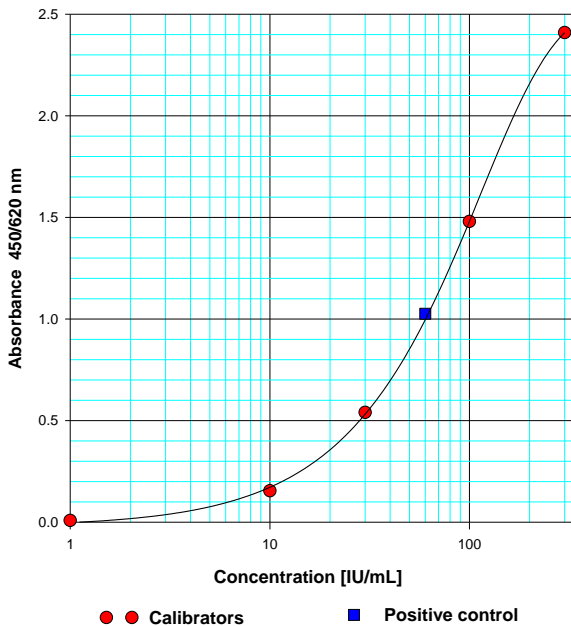

## Interference

Hemolytic, lipemic and icteric samples do not interfere when present in concentrations of up to 500 mg/dL (hemoglobin), 1000 mg/dL (lipids) and 20 mg/dL (bilirubin C and bilirubin F). Rheumatoid factors do not interfere in a concentration of up to 500 IU/mL.

## Change History

| Version           | Section         | Modifications   |
|-------------------|-----------------|---|
| 2023-03_v01_de_en | Entire document | Updating of the intended use<br>Conversion of subsections<br>Insertion of safety instructions<br>Updating "Important information" |

## References

1. Conrad, K, Schöblier, W, et al. (2015): "Autoantibodies in Systemic Autoimmune Diseases. A Diagnostic Reference", Pabst Science Publishers Lengerich, Vol. 4.
2. Guchelaar, NAD, Waling, MM, et al. (2020): "The value of anti-neutrophil cytoplasmic antibodies (ANCA) testing for the diagnosis of ANCA-associated vasculitis, a systematic review and meta-analysis", Autoimmun Rev. 20(1):102716. doi: 10.1016/j.autrev.2020.102716.
3. Peter, JB, Shoenfeld, Y (1996): "Autoantibodies", Elsevier Amsterdam.
4. Tan, EM (1989): "Antinuclear antibodies: diagnostic markers for autoimmune disease and probes for cell biology", Adv. Immunol. 44: 93-151.