## SeraSpot<sup>®</sup> ANA-17 IgG

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## SeraSpot<sup>®</sup> ANA-17 IgG

Spot immunoassay for the quantitative detection of IgG autoantibodies against 17 antinuclear and cytoplasmic antigens in serum of human origin

|            |                                    |   |           |
|------------|------------------------------------|---|-----------|
| <b>REF</b> | SP-002-17 G-S6                     |  | 48        |
| <b>REF</b> | SP-002-17 G-S12                    |  | 96        |
| <b>REF</b> | SP-002-17 G-S24                    |  | 2 x 96    |
| <b>IVD</b> | In-vitro-diagnostic medical device |   | <b>CE</b> |

Seramun Diagnostica GmbH • Sprenhagener Str. 1 • 15754 Heidesee • Germany •  
T +49 33767 791-10 • [info@seramun.com](mailto:info@seramun.com) • [www.seramun.com](http://www.seramun.com)

**IVD** In-vitro diagnostic medical device

 Country of manufacture and date of manufacture

 Keep away from sunlight

 Consult instructions for use

 Sufficient for *n* tests

**UDI** Unique device identifier

**REF** Article number

 Humidity limitation

 Temperature limit

 Biohazard

 Manufacturer

**SN** Serial number

**LOT** Batch code

 Do not reuse

 Use-by date

 Attention

## Intended Use

SeraSpot® ANA-17 IgG is an IVD test for the quantitative determination of antinuclear and cytoplasmatic autoantibodies of the IgG isotype against the antigens Mi-2, Ku, U1-snRNP, CENP-B, Nucleosome, Histone, dsDNA, Jo-1, Scl-70, Sm, P0, PCNA, La/SS-B, Ro/SS-A (52 kDa), Ro/SS-A (60 kDa), AMA-M2 and PM-Scl in serum of human origin by a laboratory professional user.

The test is used in combination with the Seramun SpotSight® plate mono / Seramun SpotSight® strip device for image acquisition and the software Seramun SpotSight® scan for image analysis.

It is intended to aid in the diagnosis of connective tissue diseases in specimen materials from patients with suspicion of a systemic autoimmune disease.

The test must not be used with specimen materials other than serum of human origin, for diagnosis, monitoring, screening, prediction, prognosis, as companion diagnostic, in the near-patient setting and by lay persons.

## Principle of the Test

SeraSpot® ANA-17 IgG is a solid phase immunoassay (spot immunoassay) using recombinant or native purified antigens printed in array format (spot array) on the bottom of wells of 96-well microtiter plates. Antibodies will bind to the immobilized antigens Mi-2, Ku, U1-snRNP, CENP-B, Nucleosome, Histone, dsDNA, Jo-1, Scl-70, Sm, P0, PCNA, La/SS-B, Ro/SS-A (52 kDa), Ro/SS-A (60 kDa), AMA-M2, and PM-Scl. After incubation and wash steps bound antibodies are detected by horseradish peroxidase (HRP)-labeled antibodies against human antibodies of IgG-isotype by substrate reaction with 3,3',5,5'-tetramethylbenzidine (TMB) and hydrogen peroxide. Blue spots indicate the formation of immune complexes. Spots range from pale to dark blue and are visible by eye.

### Detection of Specific Antibodies is performed in 3 Steps:

#### Step 1

Incubation of 1 : 101 diluted samples for 30 min at room temperature in selected wells. After incubation samples are aspirated and unbound components are removed by 3 wash cycles with diluted wash buffer.

#### Step 2

Incubation of wells with the HRP-labeled anti-human IgG for 30 min at room temperature. Removal of unbound conjugate antibodies by aspiration and 3 wash cycles with diluted wash buffer.

#### Step 3

Incubation of wells with substrate SeramunBlau® spot dark for 30 min at room temperature. The reaction is stopped by aspirating the substrate, followed by tapping the plate / strip dry onto lint-free absorbent paper. The developed arrays are to be stored light-protected until image analysis.

## Test Components (Delivery Scope)

|   |               |  | For 48 determinations  | For 96 determinations   | For 2 x 96 determinations   |
|---|---------------|--|--|---|---|
| 1 | WELLS         | <b>Microtiter plate (wells with arrays)</b><br>specific antigens and controls immobilized as spots in array layout | 6 single breakable 8-well strips in frame<br>color coding: dark purple<br>vacuum sealed with desiccant | 12 single breakable 8-well strips in frame<br>color coding: dark purple<br>vacuum sealed with desiccant | 2 x 12 single breakable 8-well strips in frame<br>color coding: dark purple<br>vacuum sealed with desiccant |
| 2 | WASHBUF (10x) | <b>Wash buffer (10x)</b><br>Seramun® Wash buffer A<br>TRIS-based buffer  | 100 mL concentrate for 1000 mL buffer<br>colorless white cap   | 100 mL concentrate for 1000 mL buffer<br>colorless white cap  | 2 x 100 mL concentrate for 1000 mL buffer each<br>colorless white cap                                       |
| 3 | DIL           | <b>Sample diluent</b><br>Seramun® Sample diluent B   | 55 mL ready-to-use solution<br>colored red black cap   | 2 x 55 mL ready-to-use solution<br>colored red black cap  | 4 x 55 mL ready-to-use solution<br>colored red black cap  |
| 4 | CONJ HRP IgG  | <b>Conjugate</b><br>anti-Human IgG-HRP conjugate (sheep)   | 8.0 mL ready-to-use solution<br>colored red red cap  | 8.0 mL ready-to-use solution<br>colored red red cap   | 2 x 8.0 mL ready-to-use solution<br>colored red red cap   |
| 5 | SUBSTR        | <b>Substrate</b><br>SeramunBlau® spot dark<br>3,3',5,5'-Tetramethylbenzidine                                       | 8.0 mL ready-to-use solution<br>colorless blue cap   | 8.0 mL ready-to-use solution<br>colorless blue cap  | 2 x 8.0 mL ready-to-use solution<br>colorless blue cap  |
| 6 | COVER         | <b>Covering film</b>   | 2 pieces   | 2 pieces  | 4 pieces  |
| 7 | SWAB          | <b>Swab</b><br>70 % (v/v) Isopropyl alcohol  | 3 x 2 pieces   | 6 x 2 pieces  | 12 x 2 pieces   |
| 8 |               | <b>Certificate of Analysis</b>   | 1 piece  | 1 piece   | 1 piece   |
| 9 |               | <b>Instructions for Use</b>  | 1 piece  | 1 piece   | 1 piece   |

## Antigens Used

| Name | Description                                   | Clinical relevance  |
|------|---|---|
| Mi-2 | Nuclear protein                               | Dermatomyositis (DM),<br>Diagnostic marker<br>Scleroderma (SSc) CREST-Syndrome  |
| Ku   | Heterodimeric subunit of DNA-dependent kinase | Scleroderma<br>Polymyositis (PM)/ Scleroderma (SSc)-Overlap-Syndrome<br>Systemic Lupus Erythematosus (SLE)<br>Dermatomyositis (DM)<br>Mixed Connective Tissue Disease (MCTD)<br>Myositis<br>Sjogren's Syndrome (SS) |

|            |   |  |
|------------|---|--|
| U1-snRNP   | Mix of three U1-specific RNP-proteins A, C und 68 kDa         | Mixed Connective Tissue Disease (MCTD),<br>Diagnostic marker<br>Mixed Connective Tissue Disease (MCTD)/ Myositis-Overlap-Syndrome<br>Systemic Lupus Erythematosus (SLE)<br>Systemic Lupus Erythematosus (SLE)/ Scleroderma (SSc)<br>Sjogren's Syndrome (SS)                        |
| CENP-B     | Kinetochores/<br>centromere B antigen protein                 | Scleroderma (SSc) CREST-Syndrom, ACR/EULAR classification criterion (2013)   |
| Nucleosome | Structural component of chromatin                             | Systemic Lupus Erythematosus (SLE)<br>Lupus Nephritis (LN)<br>Drug-Induced Lupus Erythematosus (DILE)<br>Mixed Connective Tissue Disease (MCTD)  |
| Histone    | Basic element of nucleosomes                                  | Systemic Lupus Erythematosus (SLE)<br>Drug-Induced Lupus Erythematosus (DILE)<br>Lupus Nephritis (LN)  |
| dsDNA      | Double-stranded DNA   | Systemic Lupus Erythematosus (SLE),<br>Activity and prognostic markers, ACR/EULAR* classification criterion (2019)<br>Lupus Nephritis (LN)<br>Neuropsychiatric Systemic Lupus Erythematosus (NPSLE)<br>Sjogren's Syndrome (SS)<br>Scleroderma (SSc)                                |
| Jo-1       | Histidyl-tRNA synthetase                                      | Idiopathic Autoimmune Myositides,<br>Diagnostic marker<br>Polymyositis (PM, Anti-Synthetase-Syndrome)<br>Dermatomyositis (DM)  |
| Scl-70     | DNA Topoisomerase I   | Scleroderma (SSc) CREST-Syndrom, ACR/EULAR classification criterion (2013)<br>Scleroderma (SSc) /Myositis-Overlap-Syndrome   |
| Sm         | Smith antigen/<br>ribonucleoprotein                           | Systemic Lupus Erythematosus (SLE),<br>Diagnostic marker, ACR/EULAR classification criterion (2019)<br>Lupus Nephritis (LN),<br>Prognostic marker  |
| P0         | Ribosomales Phosphoprotein                                    | Systemic Lupus Erythematosus (SLE)<br>Neuropsychiatric Systemic Lupus Erythematosus (NPSLE)<br>Lupus Nephritis (LN)  |
| PCNA       | Cyclin (36 kDa),<br>auxiliary protein of DNA polymerase delta | Systemic Lupus Erythematosus (SLE)   |
| La/SS-B    | Phosphoprotein/<br>Sjogren's syndrome antigen B               | Sjogren's Syndrome (SS),<br>Early diagnostic marker<br>Systemic Lupus Erythematosus (SLE),<br>Early diagnostic marker<br>Neonatal Lupus Erythematosus (NLE)/ Congenital Heart Block (CHB)<br>Drug-Induced Subacute Cutaneous Lupus Erythematosus (DI-SCLE)<br>Lupus Nephritis (LN) |

|                  |  |  |
|------------------|--|--|
| Ro/SS-A (52 kDa) | 52 kDa<br>Ribonucleoprotein/<br>Sjogren`s syndrome antigen A | Sjogren's Syndrome (SS),<br>Early diagnostic marker, ACR/EULAR classification criterion (2016)<br>Systemic Lupus Erythematosus (SLE),<br>Early diagnostic marker<br>Neonatal Lupus Erythematosus (NLE)/ Congenital Heart Block (CHB)<br>Drug-induced Subacute Cutaneous Lupus Erythematosus (DI-SCLE)<br>Lupus Nephritis (LN)<br>Myositis<br>Scleroderma (SSc) |
| Ro/SS-A (60 kDa) | 60 kDa<br>Ribonucleoprotein/<br>Sjogren`s syndrome antigen A | Systemic Lupus Erythematosus (SLE),<br>Early diagnostic marker<br>Drug-Induced Lupus Erythematosus (DILE)<br>Neonatal Lupus Erythematosus (NLE)<br>Sjogren's Syndrome (SS),<br>Early diagnostic marker, ACR/EULAR classification criterion (2016)<br>Scleroderma (SSc)   |
| AMA-M2           | Mitochondrial antigen subtype M2                             | Primary Biliary Cholangitis (PBC),<br>Diagnostic marker  |
| PM-Scl           | Intracellular protein complex                                | Polymyositis (PM)/ Scleroderma (SSc)-Overlap-Syndrome<br>Poly-/Dermatomyositis (DM)  |

\* ACR American College of Rheumatology, EULAR European League Against Rheumatism

### Additional Materials and Aids Required for the Test Procedure

Adjustable single-channel micro-pipettes • adjustable 8-channel micro-pipette or multi-pipette • reagent container for multi-channel micro-pipettes • measuring cylinder • test tubes for sample dilution • test tube rack • washer for 96-well microtiter plates • laboratory system: Seramun SpotSight® plate mono / strip scanner with evaluation software Seramun SpotSight® scan • deionized water • lint-free absorbent paper • stop watch • collecting devices for infectious material • opaque cover (substrate reaction)

## Important Information

**This device is for *in-vitro* diagnostic use only.** The kit may be performed by a laboratory professional user only.

Follow the instructions carefully. The shelf life specified must be observed. Do not use reagents from damaged packages or bottles. Do not use reagents from other manufacturers.

**Mixing of test kit components of different lots is only allowed for sample diluent, wash buffer and substrate.**

All serious incidents occurring in relation with SeraSpot® ANA-17 IgG must be reported to the manufacturer and the competent authority of the EU member state in which the user and/or the patient are located.

### Information on Assay Procedure

All reagents should be stored at 2...8 °C. Bring all components to room temperature before use.

For larger sample series, pipetting reagents from liquid reservoirs using a multi-channel pipette is recommended to avoid time delays. Reagents that appear contaminated should not be used.

Avoid time delays when dispensing reagents.

Substrate must be protected from direct light!

Incorrect dilution of samples, incorrect washing of wells and incorrect timing may lead to erroneous results.

**Air bubbles** generated by forceful pipetting of samples and/or reagents may cause uneven signal formation of arrays. These arrays cannot be evaluated.

Damaged arrays, e.g. by **scratching** the bottom of the well with pipet tips or washer needles, are not suitable for evaluation.

Before taking images of the wells make sure to remove particles or fibers which may adhere to the bottom side of the wells using the swab provided in the kit.

Images of dried spots may appear more intense, which can lead to slight deviations of the measured values when scanned repeatedly. Assessment of the individual parameters is not changed.

### Workplace Requirements

Processing spot immunoassays requires a clean workplace. Adhesion of fibers to the plastic surface of the wells should be avoided. Fibers may cause interferences when taking images of arrays and cause erroneous results. Workplace and kit components should not be exposed to direct sunlight.

## Safety Instructions

Reagents must not be swallowed. Contact with skin or mucous membranes should be avoided. Some reagents may contain biocides as preservative. Handle all components and patient samples as if potentially hazardous and infectious.

Additional information may be taken from the Material Safety Data Sheet.

Product contains the following hazardous component/-s:

|                      |  |   |
|----------------------|--|---|
| <b>WELLS</b>         | -  | Contains material of animal origin.   |
| <b>WASHBUF (10x)</b> | EUH208<br>EUH210   | Contains reaction mass of 5-chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.<br>Safety data sheet available on request.  |
| <b>DIL</b>           | EUH208<br>EUH210<br>-  | Contains reaction mass of 5-chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.<br>Safety data sheet available on request.<br>Contains material of animal origin.   |
| <b>CONJ HRP IgG</b>  | Hazard components<br>EUH208<br>H360D<br>P202<br>P280<br>P308+P313<br>P501<br>-           | N-methyl-2-pyrrolidone; 1-methyl-2-pyrrolidone<br>Contains reaction mass of 5-chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.<br>Restricted to professional users.<br>May damage the unborn child.<br>Do not handle until all safety precautions have been read and understood.<br>Wear protective gloves / protective clothing / eye protection / face protection.<br>IF exposed or concerned: Get medical advice / attention.<br>Dispose of contents / container in accordance with local regulations.<br>Contains material of animal origin. |
| <b>SWAB</b>          | Hazard components<br>H225<br>H319<br>H336<br>P210<br>P280<br>P304+P340<br>P305+P351+P338 | Propan-2-ol; isopropyl alcohol; isopropanol<br>Highly flammable liquid and vapour.<br>Causes serious eye irritation.<br>May cause drowsiness or dizziness.<br>Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.<br>Wear protective gloves / protective clothing / eye protection / face protection / hearing protection.<br>IF INHALED: Remove person to fresh air and keep comfortable for breathing.<br>IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.                       |

### Limitations of the Procedure

Result interpretation should always consider clinical findings. Individual cases may require repeated investigations of samples taken at intervals of several weeks.

Impurities and cross contamination of reagents or samples by fungi or bacteria can lead to false positive as well as false negative results.

### Interference

In rare cases, specimens may contain antibodies to BSA (bovine serum albumin) and / or AGE (advanced glycation end products), which may cause nonspecific reactions, so that the test result is read as "not available" ("n.a.") by the Seramun SpotSight® scan software.

## Sample Treatment

### Sample Collection

Collect serum of human origin in suitable sampling container.

### Sample Shelf Life and Storage

Samples can be stored at 2...8 °C for a maximum of 7 days. For longer periods samples have to be stored at < -15 °C. Repeated freezing and thawing of samples should be avoided!

### Sample Preparation

Bring samples to room temperature before use. Homogeneity should be ensured by shaking components briefly.

Samples have to be diluted 1 : 101 (v/v) with sample diluent.

Example: 10 µL sample and 1000 µL sample diluent.

## Reagent Treatment

### Reagent Shelf Life and Storage

The complete test kit with sealed reagent bottles and microtitration strips is stable until the printed expiration date when stored at 2...8 °C. All opened test kit components are stable for up to 2 months when stored properly at 2...8 °C. The diluted wash buffer can be stored at 2...8 °C for up to 1 month.

### Reagent Preparation

Allow all components to reach **room temperature** prior to use. The **microtiter plate** is vacuum sealed with desiccant. The plate consists of a frame and strips with breakable wells. Allow the sealed plate to reach room temperature before opening. Unused wells should be stored at 2...8 °C and protected from moisture in the original cover carefully resealed.

Dilute **wash buffer (10x)** 1 : 10 with deionized water.

Example: 10 mL wash buffer (10x) + 90 mL deionized water. The prepared wash buffer must be thoroughly mixed before use.

The **substrate** must be protected from direct light. The test should not be performed with a substrate solution that is colored dark or contains colored precipitates.

## Assay Procedure

Performance at room temperature (RT, 18...25 °C). Bring **all** ready-to-use test reagents, prepared wash buffer and microplate to **room temperature**. Mix all reagents before use by gently shaking, avoid foaming.

The aspiration and washing steps can be carried out manually or by means of a microtiter plate washer.

### Important Notes on Test Procedure:

**Avoid** mechanical contact (**scratching**) on the bottom of the wells with pipette tips or washer needles. This will irreparably damage the array!

All liquid reagents (sample, conjugate and substrate) have to be pipetted **without** causing **air bubbles** in the wells.


1. Provide the required number of wells **WELLS** according to the number of samples.
2. Preparation of working dilution of the sample: 1 : 101 dilution  
e.g. 10 µL sample to 1000 µL sample diluent **DIL**
3. Pipette **100 µL** diluted sample per well.
4. Cover plate with covering film **COVER** and incubate for **30 min** at RT.
5. Remove liquid, then wash wells **3x** using **400 µL** diluted wash buffer per well,  
Tap plate dry onto lint-free absorbent paper.
6. **Add 50 µL** of the conjugate **CONJ HRP IgG** (anti-Human IgG-HRP) to each well.
7. Cover plate with covering film **COVER** and incubate for **30 min** at RT.
8. Remove liquid, then wash wells **3x** using **400 µL** diluted wash buffer per well,  
Tap plate dry onto lint-free absorbent paper.
9. **Add 50 µL** of the substrate **SUBSTR** to each well.
10. Cover plate and protect from light. Incubate for **30 min** at RT.
11. Remove liquid. Tap plate dry onto lint-free absorbent paper.
12. Clean bottom of wells with swab **SWAB** shortly before scanning the images.
13. **Take images** using the Seramun SpotSight® plate mono / strip und evaluate the results with the Seramun SpotSight® scan software.

If image acquisition is performed with the scanner Seramun SpotSight® strip, transfer single 8-well microtiter plate strips to the scanner's strip holder.

After aspiration of the substrate the color of developed spots is stable for 24 h when the plate is stored protected from light.

## Evaluation of Results

### Arraylayout

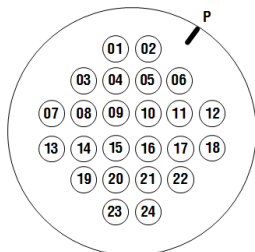### Antigens

|    |                  |
|----|------------------|
| 01 | Mi-2             |
| 02 | Ku               |
| 07 | U1-snRNP         |
| 08 | CENP-B           |
| 09 | Nucleosome       |
| 10 | Histone          |
| 12 | dsDNA            |
| 13 | Jo-1             |
| 14 | Scl-70           |
| 15 | Sm               |
| 16 | P0               |
| 18 | PCNA             |
| 19 | La/SS-B          |
| 20 | Ro/SS-A (52 kDa) |
| 21 | Ro/SS-A (60 kDa) |
| 23 | AMA-M2           |
| 24 | PM-Scl           |

### Controls

|    |                                    |
|----|------------------------------------|
| 03 | Positive control (PC)              |
| 04 | Negative control (NC), 0 rel.Units |
| 05 | Cut-off control (CO), 30 rel.Units |
| 06 | Reference 3 (R3), 60 rel.Units     |
| 11 | Reference 2 (R2), 100 rel.Units    |
| 17 | Reference 1 (R1), 300 rel.Units    |
| 22 | Serum control (SC)                 |
| P  | Well position marker               |

### Quantitative evaluation

#### Validity Criteria for the Test

The spots NC, CO, R1 ... R3 are used to create a reference curve (4-parameter non-linear regression) to calculate the staining intensities of the antigen spots in relative units (rel.Units) or in international units (IU) for dsDNA.

SeraSpot® ANA-17 IgG includes the following control spots:

1. Positive control (PC). Intense spot, stained darker than cut-off control. Always stained.
2. Cut-off control (CO). Weakly stained spot. Used for result evaluation of parameter specific signals.
3. Negative control (NC). Pale spot with intensity lower than cut-off control.
4. Serum control (SC). Intense spot, always stained in presence of sample. Absence of spot indicates absence of sample.
5. Reference 1 (R1) corresponds to 300 rel.Units. Intense spot, always stained.
6. Reference 2 (R2) corresponds to 100 rel.Units. Weaker staining than R1, always stained.
7. Reference 3 (R3) corresponds to 60 rel.Units. Weaker staining than R2, always stained.

The test cannot be evaluated if one of the validity criteria listed in points 1. to 7. is not met.

If the above mentioned validity criteria are not met, test should be repeated strictly following the test procedure (incubation times and temperatures, sample and wash buffer dilution, wash steps, etc.). In case of repeated failure of the validity criteria contact the manufacturer.

## Interpretation of Results

Evaluation of the test has to be performed by using the Seramun SpotSight® plate mono / strip and in combination with the Seramun SeraSpot® scan software.

Results are interpreted as follows:

| Evaluation | Conditions   |
|------------|--|
| Positive   | Color intensity of one or more antigen spots > cut-off control |
| Negative   | Color intensity of the antigen spots ≤ cut-off control         |

## Performance Characteristics

### Precision

Samples with known antibody reactivity were examined in the SeraSpot® ANA-17 IgG and the rel.Units were determined. These values were then used to determine the coefficients of variation (CV) as a measure of the precision within a test run (intra-assay CV), between different test runs (inter-assay CV) and between different test lots (lot-to-lot CV).

| Antigen    | Intra-assay coefficient of variation       |        | Inter-assay coefficient of variation  |        | Lot-to-lot coefficient of variation   |        |
|------------|--|--------|---|--------|---|--------|
|            | $\bar{x}$ rel.Units<br>n=40                | CV [%] | $\bar{x}$ rel.Units<br>n=80   | CV [%] | $\bar{x}$ rel.Units<br>n=240  | CV [%] |
| Mi-2       | 92.4                                       | 10.1   | 96.7  | 14.6   | 99.9  | 14.1   |
| Ku         | 20.7                                       | 10.0   | 39.8  | 15.4   | 39.5  | 16.0   |
| U1-snRNP   | 64.9                                       | 8.9    | 55.8  | 9.0    | 56.3  | 15.5   |
| CENP-B     | 39.0                                       | 8.1    | 37.4  | 12.3   | 35.3  | 12.7   |
| Nucleosome | 43.6                                       | 11.9   | 48.5  | 19.3   | 48.9  | 24.2   |
| Histone    | 67.6                                       | 8.8    | 57.3  | 14.8   | 55.6  | 12.3   |
| dsDNA      | 63.3                                       | 7.8    | 72.9  | 11.2   | 72.4  | 11.7   |
| Jo-1       | 48.7                                       | 5.4    | 47.4  | 10.5   | 49.1  | 15.2   |
| Scl-70     | 37.1                                       | 11.6   | 40.4  | 15.9   | 42.3  | 16.4   |
| Sm         | 77.4                                       | 10.5   | 93.2  | 12.8   | 91.8  | 13.3   |
| P0         | 57.9                                       | 7.9    | 48.7  | 10.1   | 54.6  | 21.0   |
| PCNA       | 85.0                                       | 13.9   | 100.1   | 13.3   | 100.6   | 24.1   |
| La/SS-B    | 54.7                                       | 11.9   | 53.1  | 10.2   | 52.8  | 13.9   |
| Ro/SS-A 52 | 68.0                                       | 9.3    | 84.2  | 12.9   | 88.9  | 19.6   |
| Ro/SS-A 60 | 96.3                                       | 13.3   | 130.7   | 15.9   | 143.6   | 18.7   |
| AMA-M2     | 81.2                                       | 12.9   | 84.4  | 13.0   | 75.7  | 24.1   |
| PM-Scl     | 43.5                                       | 13.1   | 47.9  | 12.0   | 48.8  | 12.5   |
| Procedure  | 1 operator<br>40 determinations<br>1 batch |        | 5 operators<br>2 determinations<br>2 testings per day<br>20 days<br>1 batch |        | 5 operators<br>2 determinations<br>2 testings per day<br>20 days<br>3 batches |        |

### Determination of the Cut-off Value

Data from sensitivity and specificity studies compared to reference tests were used to determine the value for the cut-off control by ROC analysis (Receiver Operating Characteristics).

The cut-off range is determined specifically for each test.

## Interfering Substances

All tested substances do not have a significant impact on the test results even if present in elevated concentrations in the serum: 20 mg/dL bilirubin C and F each (simulation of icteric samples), 500 mg/dL hemoglobin (simulation of hemolytic samples), 1000 mg/dL lipids (triglycerides) (simulation of lipemic samples) and 500 IU/mL rheumatoid factor.

## Sensitivity

Pre-characterized samples were analyzed to determine the sensitivity and specificity of SeraSpot® ANA-17 IgG in comparison with IgG reference tests. Samples with discrepant results were retested in a second reference test.

| Mi-2          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 9              | 9        |
|               | negative | 0              | 42       |
| Samples total |          | 9              | 51       |
| Sensitivity:  |          | 100 %          |          |

| Ku            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 5              | 0        |
|               | negative | 0              | 55       |
| Samples total |          | 5              | 55       |
| Sensitivity:  |          | 100 %          |          |

| U1-snRNP      |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 32             | 0        |
|               | negative | 3              | 12       |
| Samples total |          | 35             | 12       |
| Sensitivity:  |          | 91.4 %         |          |

| CENP-B        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 18             | 2        |
|               | negative | 0              | 92       |
| Samples total |          | 18             | 94       |
| Sensitivity:  |          | 100 %          |          |

| Nucleosome    |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 37             | 3        |
|               | negative | 9              | 54       |
| Samples total |          | 46             | 57       |
| Sensitivity:  |          | 80.4 %         |          |

| Histone       |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 36             | 12       |
|               | negative | 2              | 53       |
| Samples total |          | 38             | 65       |
| Sensitivity:  |          | 94.7 %         |          |

| dsDNA         |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 70             | 8        |
|               | negative | 2              | 23       |
| Samples total |          | 72             | 31       |
| Sensitivity:  |          | 97.2 %         |          |

| Jo-1          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 14             | 3        |
|               | negative | 1              | 42       |
| Samples total |          | 15             | 45       |
| Sensitivity:  |          | 93.3 %         |          |

| Scl-70        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 59             | 3        |
|               | negative | 5              | 45       |
| Samples total |          | 64             | 48       |
| Sensitivity:  |          | 92.2 %         |          |

| Sm            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 8              | 1        |
|               | negative | 1              | 93       |
| Samples total |          | 9              | 94       |
| Sensitivity:  |          | 88.9 %         |          |

| P0            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 6              | 5        |
|               | negative | 0              | 92       |
| Samples total |          | 6              | 97       |
| Sensitivity:  |          | 100 %          |          |

| PCNA          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 4              | 0        |
|               | negative | 0              | 103      |
| Samples total |          | 4              | 103      |
| Sensitivity:  |          | 100 %          |          |

| La/SS-B       |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 12             | 5        |
|               | negative | 3              | 83       |
| Samples total |          | 15             | 88       |
| Sensitivity:  |          | 80.0 %         |          |

| Ro/SS-A (52 kDa) |          | Reference test |          |
|------------------|----------|----------------|----------|
|                  |          | positive       | negative |
| SeraSpot®        | positive | 27             | 0        |
|                  | negative | 0              | 76       |
| Samples total    |          | 27             | 76       |
| Sensitivity:     |          | 100 %          |          |

| Ro/SS-A (60 kDa) |          | Reference test |          |
|------------------|----------|----------------|----------|
|                  |          | positive       | negative |
| SeraSpot®        | positive | 42             | 0        |
|                  | negative | 2              | 59       |
| Samples total    |          | 44             | 59       |
| Sensitivity:     |          | 95.5 %         |          |

| AMA-M2        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 15             | 4        |
|               | negative | 0              | 77       |
| Samples total |          | 15             | 81       |
| Sensitivity:  |          | 100 %          |          |

| PM/Sci        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 6              | 0        |
|               | negative | 1              | 53       |
| Samples total |          | 7              | 53       |
| Sensitivity:  |          | 85.7 %         |          |

### Specificity

Blood donor samples were examined for the specificity of SeraSpot® ANA-17 IgG. Samples with positive results were retested in a reference test.

| Mi-2          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 12       |
|               | negative | 0              | 188      |
| Samples total |          | 0              | 200      |
| Specificity:  |          | 94.0 %         |          |

| Ku            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 2        |
|               | negative | 0              | 198      |
| Samples total |          | 0              | 200      |
| Specificity:  |          | 99.0 %         |          |

| U1-snRNP      |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 18       |
|               | negative | 0              | 182      |
| Samples total |          | 0              | 200      |
| Specificity:  |          | 91.0 %         |          |

| CENP-B        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 2        |
|               | negative | 0              | 198      |
| Samples total |          | 0              | 200      |
| Specificity:  |          | 99.0 %         |          |

| Nucleosome    |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 11       |
|               | negative | 0              | 189      |
| Samples total |          | 0              | 200      |
| Specificity:  |          | 94.5 %         |          |

| Histone       |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 3              | 10       |
|               | negative | 0              | 187      |
| Samples total |          | 3              | 197      |
| Specificity:  |          | 94.9 %         |          |

| dsDNA         |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 2        |
|               | negative | 0              | 198      |
| Samples total |          | 0              | 200      |
| Specificity:  |          | 99.0 %         |          |

| Jo-1          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 1        |
|               | negative | 0              | 199      |
| Samples total |          | 0              | 200      |
| Specificity:  |          | 99.5 %         |          |

| Sci-70        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 7        |
|               | negative | 0              | 193      |
| Samples total |          | 0              | 200      |
| Specificity:  |          | 96.5 %         |          |

| Sm            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 1        |
|               | negative | 0              | 199      |
| Samples total |          | 0              | 200      |
| Specificity:  |          | 99.5 %         |          |

| P0            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 1        |
|               | negative | 0              | 199      |
| Samples total |          | 0              | 200      |
| Specificity:  |          | 99.5 %         |          |

| PCNA          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 200      |
| Samples total |          | 0              | 200      |
| Specificity:  |          | 100 %          |          |

| La/SS-B       |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 2              | 7        |
|               | negative | 0              | 191      |
| Samples total |          | 2              | 198      |

Specificity: 96.5 %

| Ro/SS-A (52 kDa) |          | Reference test |          |
|------------------|----------|----------------|----------|
|                  |          | positive       | negative |
| SeraSpot®        | positive | 2              | 2        |
|                  | negative | 0              | 196      |
| Samples total    |          | 2              | 198      |

Specificity: 99.0 %

| Ro/SS-A (60 kDa) |          | Reference test |          |
|------------------|----------|----------------|----------|
|                  |          | positive       | negative |
| SeraSpot®        | positive | 0              | 1        |
|                  | negative | 0              | 199      |
| Samples total    |          | 0              | 200      |

Specificity: 99.5 %

| AMA-M2        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 11       |
|               | negative | 0              | 189      |
| Samples total |          | 0              | 200      |

Specificity: 94.5 %

| PM-Scl        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 6        |
|               | negative | 0              | 194      |
| Samples total |          | 0              | 200      |

Specificity: 97.0 %

### Cross Reactivity

Potential cross-reactive samples were tested in the SeraSpot® ANA-17 IgG and re-tested in a reference test. Samples with discrepant results were re-tested in a second reference test. Results obtained with the second reference test were used for the final status determination of the discrepant samples.

### Samples for patients with rheumatoid arthritis

A collection of 113 samples from patients suffering from rheumatoid arthritis with a positive anti-CCP (n=30) or RF (n=83) serology (reference test) were tested in the SeraSpot® ANA-17 IgG.

| Mi-2          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 9              | 7        |
|               | negative | 0              | 97       |
| Samples total |          | 9              | 104      |

Specificity: 93.3 %

| Ku            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 6              | 4        |
|               | negative | 0              | 103      |
| Samples total |          | 6              | 107      |

Specificity: 96.3 %

| U1-snRNP      |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 15       |
|               | negative | 0              | 98       |
| Samples total |          | 0              | 113      |

Specificity: 86.7 %

| CENP-B        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 2              | 1        |
|               | negative | 0              | 110      |
| Samples total |          | 2              | 111      |

Specificity: 99.1 %

| Nucleosome    |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 2              | 2        |
|               | negative | 2              | 107      |
| Samples total |          | 4              | 109      |

Specificity: 98.2 %

| Histone       |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 9              | 14       |
|               | negative | 0              | 90       |
| Samples total |          | 9              | 104      |

Specificity: 86.5 %

| dsDNA         |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 2              | 3        |
|               | negative | 0              | 108      |
| Samples total |          | 2              | 111      |

Specificity: 97.3 %

| Jo-1          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 113      |
| Samples total |          | 0              | 113      |

Specificity: 100 %

| Sci-70        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 7              | 2        |
|               | negative | 0              | 104      |
| Samples total |          | 7              | 106      |

Specificity: 98.1 %

| Sm            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 1        |
|               | negative | 0              | 112      |
| Samples total |          | 0              | 113      |
| Specificity:  |          | 99.1 %         |          |

| P0            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 1              | 1        |
|               | negative | 0              | 111      |
| Samples total |          | 1              | 112      |
| Specificity:  |          | 99.1 %         |          |

| PCNA          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 2              | 0        |
|               | negative | 0              | 111      |
| Samples total |          | 2              | 111      |
| Specificity:  |          | 100 %          |          |

| La/SS-B       |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 7        |
|               | negative | 0              | 106      |
| Samples total |          | 0              | 113      |
| Specificity:  |          | 93.8 %         |          |

| Ro/SS-A (52 kDa) |          | Reference test |          |
|------------------|----------|----------------|----------|
|                  |          | positive       | negative |
| SeraSpot®        | positive | 5              | 1        |
|                  | negative | 0              | 107      |
| Samples total    |          | 5              | 108      |
| Specificity:     |          | 99.1 %         |          |

| Ro/SS-A (60 kDa) |          | Reference test |          |
|------------------|----------|----------------|----------|
|                  |          | positive       | negative |
| SeraSpot®        | positive | 2              | 1        |
|                  | negative | 0              | 110      |
| Samples total    |          | 2              | 111      |
| Specificity:     |          | 99.1 %         |          |

| AMA-M2        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 11             | 4        |
|               | negative | 0              | 98       |
| Samples total |          | 11             | 102      |
| Specificity:  |          | 96.1 %         |          |

| PM-Scl        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 2              | 0        |
|               | negative | 0              | 111      |
| Samples total |          | 2              | 111      |
| Specificity:  |          | 100 %          |          |

**Epstein-Barr-virus (EBV) antibody positive samples**

EBV antibody positive samples (reference test) were tested in SeraSpot® ANA-17 IgG.

| Mi-2          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 7        |
|               | negative | 0              | 15       |
| Samples total |          | 0              | 22       |

Specificity: 68.2 %

| Ku            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 22       |
| Samples total |          | 0              | 22       |

Specificity: 100 %

| U1-snRNP      |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 22       |
| Samples total |          | 0              | 22       |

Specificity: 100 %

| CENP-B        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 2              | 0        |
|               | negative | 0              | 20       |
| Samples total |          | 2              | 20       |

Specificity: 100 %

| Nucleosome    |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 1        |
|               | negative | 0              | 21       |
| Samples total |          | 0              | 22       |

Specificity: 95.5 %

| Histone       |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 5        |
|               | negative | 0              | 17       |
| Samples total |          | 0              | 22       |

Specificity: 77.3 %

| dsDNA         |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 1              | 1        |
|               | negative | 0              | 20       |
| Samples total |          | 1              | 21       |

Specificity: 95.2 %

| Jo-1          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 22       |
| Samples total |          | 0              | 22       |

Specificity: 100 %

| Scl-70        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 1        |
|               | negative | 0              | 21       |
| Samples total |          | 0              | 22       |

Specificity: 95.5 %

| Sm            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 1        |
|               | negative | 0              | 21       |
| Samples total |          | 0              | 22       |

Specificity: 95.5 %

| P0            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 22       |
| Samples total |          | 0              | 22       |

Specificity: 100 %

| PCNA          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 22       |
| Samples total |          | 0              | 22       |

Specificity: 100 %

| La/SS-B       |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 3        |
|               | negative | 0              | 19       |
| Samples total |          | 0              | 22       |

Specificity: 86.4 %

| Ro/SS-A (52 kDa) |          | Reference test |          |
|------------------|----------|----------------|----------|
|                  |          | positive       | negative |
| SeraSpot®        | positive | 0              | 0        |
|                  | negative | 0              | 22       |
| Samples total    |          | 0              | 22       |

Specificity: 100 %

| Ro/SS-A (60 kDa) |          | Reference test |          |
|------------------|----------|----------------|----------|
|                  |          | positive       | negative |
| SeraSpot®        | positive | 0              | 0        |
|                  | negative | 0              | 22       |
| Samples total    |          | 0              | 22       |

Specificity: 100 %

| AMA-M2        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 3        |
|               | negative | 0              | 19       |
| Samples total |          | 0              | 22       |

Specificity: 86.4 %

| PM-Scl        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 1        |
|               | negative | 0              | 21       |
| Samples total |          | 0              | 22       |

Specificity: 95.5 %

### Cytomegalovirus (CMV) antibody positive samples

CMV antibody positive samples (reference test) were tested in SeraSpot® ANA-17 IgG.

| Mi-2          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 4        |
|               | negative | 0              | 23       |
| Samples total |          | 0              | 27       |

Specificity: 85.2 %

| Ku            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 27       |
| Samples total |          | 0              | 27       |

Specificity: 100 %

| U1-snRNP      |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 3        |
|               | negative | 0              | 24       |
| Samples total |          | 0              | 27       |

Specificity: 88.9 %

| CENP-B        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 27       |
| Samples total |          | 0              | 27       |

Specificity: 100 %

| Nucleosome    |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 2        |
|               | negative | 0              | 25       |
| Samples total |          | 0              | 27       |

Specificity: 92.6 %

| Histone       |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 1              | 26       |
| Samples total |          | 1              | 26       |

Specificity: 100 %

| dsDNA         |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 1              | 1        |
|               | negative | 0              | 25       |
| Samples total |          | 1              | 26       |

Specificity: 96.2 %

| Jo-1          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 27       |
| Samples total |          | 0              | 27       |

Specificity: 100 %

| Scl-70        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 2        |
|               | negative | 0              | 25       |
| Samples total |          | 0              | 27       |

Specificity: 92.6 %

| Sm            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 27       |
| Samples total |          | 0              | 27       |

Specificity: 100 %

| P0            |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 27       |
| Samples total |          | 0              | 27       |

Specificity: 100 %

| PCNA          |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 27       |
| Samples total |          | 0              | 27       |

Specificity: 100 %

| La/SS-B       |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 3        |
|               | negative | 0              | 24       |
| Samples total |          | 0              | 27       |

Specificity: 88.9 %

| Ro/SS-A (52 kDa) |          | Reference test |          |
|------------------|----------|----------------|----------|
|                  |          | positive       | negative |
| SeraSpot®        | positive | 0              | 1        |
|                  | negative | 0              | 26       |
| Samples total    |          | 0              | 27       |

Specificity: 96.3 %

| Ro/SS-A (60 kDa) |          | Reference test |          |
|------------------|----------|----------------|----------|
|                  |          | positive       | negative |
| SeraSpot®        | positive | 0              | 0        |
|                  | negative | 0              | 27       |
| Samples total    |          | 0              | 27       |

Specificity: 100 %

| AMA-M2        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 2        |
|               | negative | 0              | 25       |
| Samples total |          | 0              | 27       |

Specificity: 92.6 %

| PM-Scl        |          | Reference test |          |
|---------------|----------|----------------|----------|
|               |          | positive       | negative |
| SeraSpot®     | positive | 0              | 0        |
|               | negative | 0              | 27       |
| Samples total |          | 0              | 27       |

Specificity: 100 %

## Change History

| Version           | Section                     | Modifications   |
|-------------------|-----------------------------|---|
| 2023-01_v01_DE_EN | Entire document             | Use of new IFU template<br>Updating of the intended use<br>Conversion of the subsections<br>Insertion of safety instructions<br>Removal of pipetting scheme |
|                   | Important Information       | Insertion note on dealing with serious incidents  |
|                   | Entire document             | Updating of wording and correction of spelling mistakes   |
|                   | Performance Characteristics | Correction of performance characteristics (Sm, Jo-1)  |

## References

1. Conrad, K., Schößler, W., Hiepe, F., Autoantikörper bei systemischen Autoimmunerkrankungen Pabst Science Publishers Lengerich u.a. 2006, 2. Aufl.
2. Peter, J.B., Shoenfeld Y. (Eds), Autoantibodies Elsevier Amsterdam u.a. 1996
3. Tan, E.M., Antinuclear antibodies: diagnostic markers for autoimmune disease and probes for cell biology, Adv. Immunol. 44, 1989: 93-151